

SPEEDSNARE[®]
Suture Passing Device

MINIMALLY INVASIVE

1.8mm shaft is an effective combination of sturdiness and size.

Pass and retrieve functionality allows single portal suture passing technique.

Leaves a **small footprint** while designed to resist breakage.

Single piece suture hook designed for durability.

MULTIPLE APPLICATIONS WITH MULTIPLE TIP CONFIGURATIONS

- Left, right, and up bends to address various hip and shoulder labral pathologies
- 30° and 60° up bends allow retrograde rotator cuff suture passing techniques and also addresses subscap applications.
- The SpeedSnare Straight device can be used in straight tunnel suture passing, for tendon transfer applications, and to shuttle suture through grafts in superior capsular reconstruction cases.

30° Left

Straight


60° Up

30° Up

30° Right

30° Up Reverse


60° Up Reverse

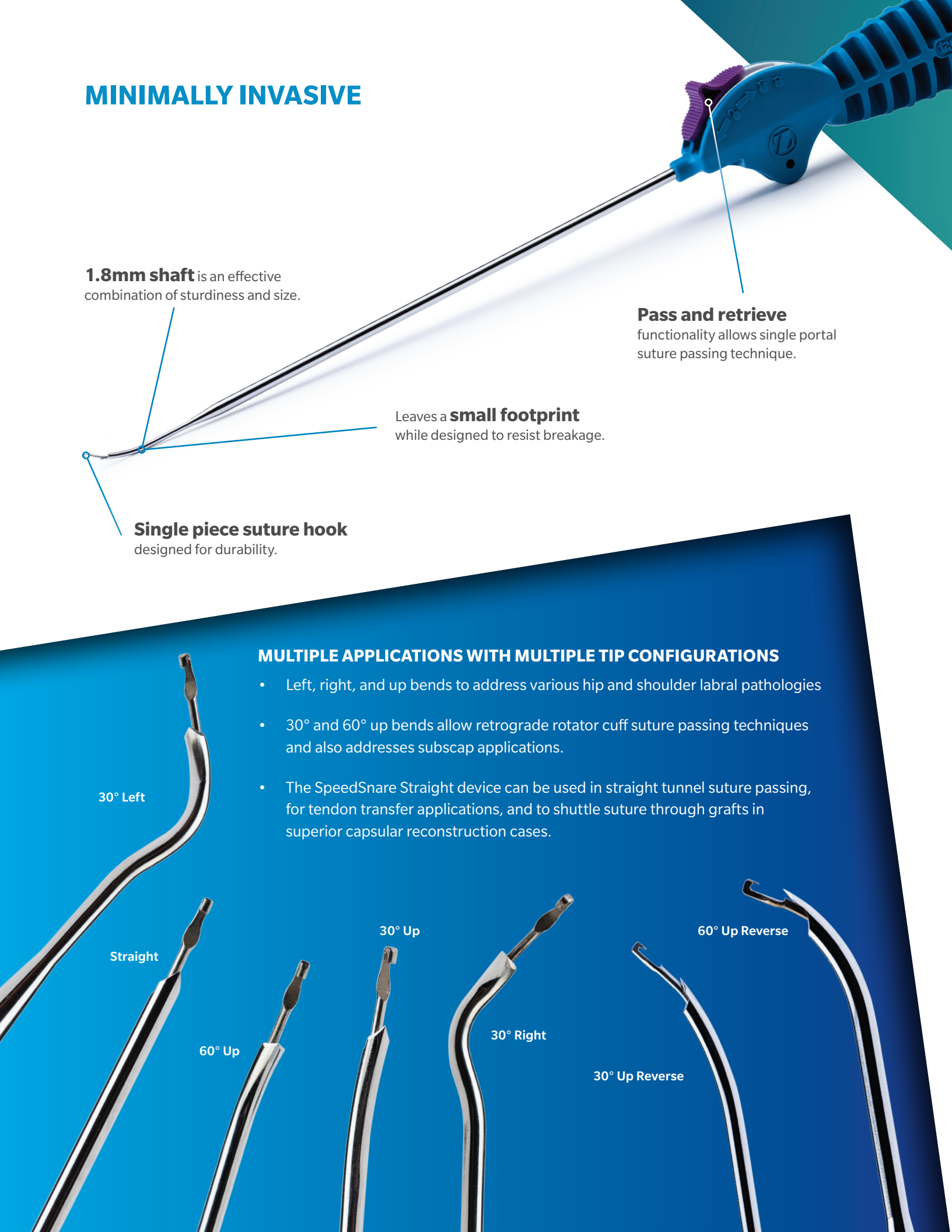

PASS IT YOUR WAY

SUTURE LOCKED AND SUTURE SLIDING POSITIONS

- Pass and retrieve suture or suture tape in **one step** with **one portal**, eliminating the need for additional suture management tools.
- Device locks suture or allows suture to freely slide during passing, depending on desired technique.

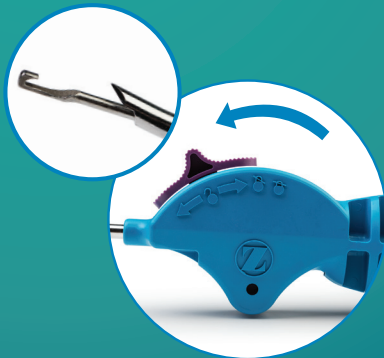


Suture Retrieval



Suture Sliding



Suture Locked

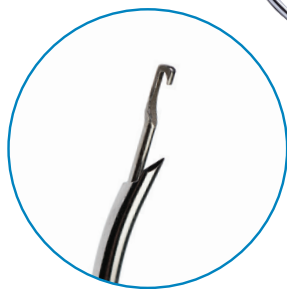

Sharp tip of device is on the **inside** of the curve.



Sharp tip of device is on the **outside** of the curve.

SPEEDSNARE AND SPEEDSNARE REVERSE

- SpeedSnare Suture Passing Devices are designed with the sharp tip of the shaft on the inside of the curve. The orientation of the sharp tip and the suture grasper facilitates optimal suture grasping.
- SpeedSnare Reverse Suture Passing Devices are designed with the sharp tip of the shaft on the outside of the curve to facilitate tissue penetration.



Ordering Information

Description	Type	Part Number
SpeedSnare Suture Passer	30° Up	110030129
	60 Up	110030130
	30° Right	110030131
	30° Left	110030132
	Straight	110030133
	30° Up Reverse	110030140
	60° Up Reverse	110030141

All content herein is protected by copyright, trademarks and other intellectual property rights, as applicable, owned by or licensed to Zimmer Biomet or its affiliates unless otherwise indicated, and must not be redistributed, duplicated or disclosed, in whole or in part, without the express written consent of Zimmer Biomet.

This material is intended for health care professionals. Distribution to any other recipient is prohibited.

For product information, including indications, contraindications, warnings, precautions, potential adverse effects and patient counseling information, see the package insert and www.zimmerbiomet.com.

Check for country product clearances and reference product specific instructions for use.

Not for distribution in France.

©2020 Zimmer Biomet

Distributor

Biomet Sports Medicine
56 East Bell Drive
P.O. Box 587
Warsaw, Indiana 46581
USA



ZIMMER BIOMET

Your progress. Our promise.®



Legal Manufacturer

T.A.G Medical Products
Corporation LTD
Kibbutz Gaaton 2531000
Israel

www.zimmerbiomet.com

CE 0459

CE mark on a surgical technique is not valid unless there is a CE mark on the product label.

2905.1-GLBL-en-REV0120