




Persona™ Adjustable Resection Tower 42-5099-008-00

Cleaning Instructions

Rotate Dial

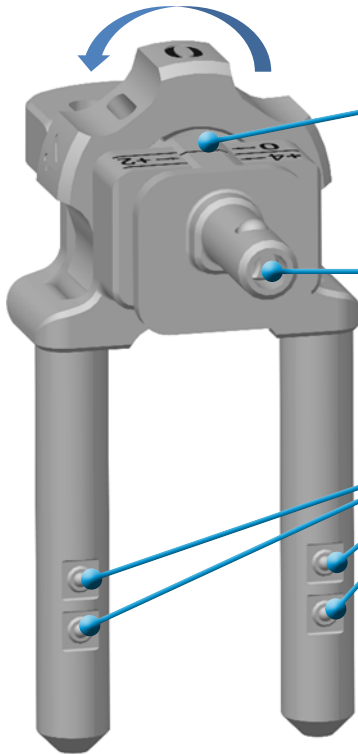



1 Flush window while rotating dial

2 Flush Bore


3 Flush 4 Ball Plungers

4 Flush 3 Ball Plungers (rotate dial)

5 Flush Spring Cavity (top and bottom)

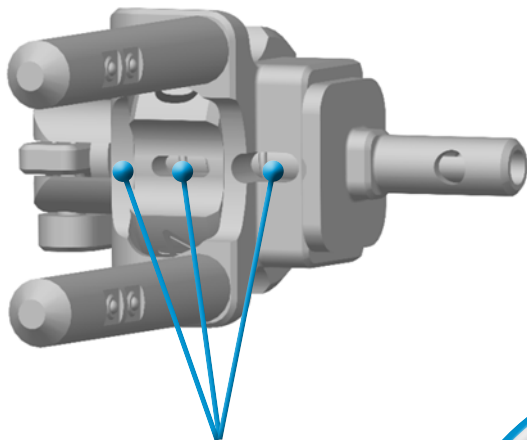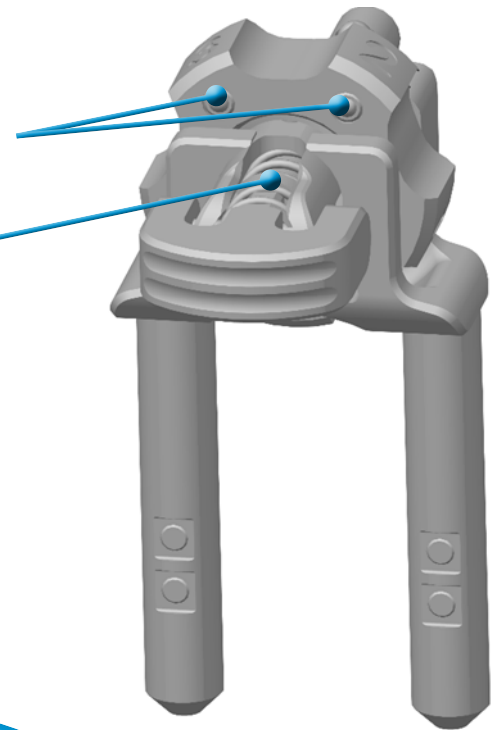

6 Flush Spring Cavity (rotate dial)

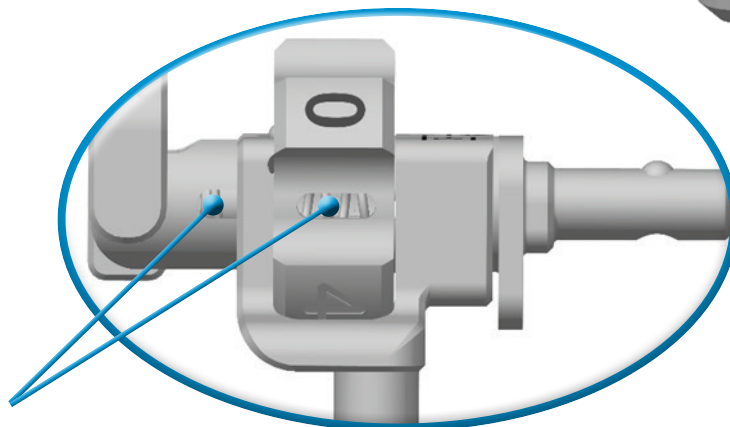

7 Flush Spring Cavity (rotate dial; both sides)

97-5026-012-00 MC0000099147/00 Printed in USA ©2013 Zimmer, Inc.